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Instructions for use

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The seat stabilizing back
and head, Multiseat Stabilo
System car seat

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Thank you for choosing our product. The NOEL Żebrowski sp. j. has designed and manufactured the seat stabilizing back and head, Multiseat Stablio System car seat to the highest safety and functional use standards. We are confident that our product will meet your highest requirements and expectations. These instructions for use are an integral part of the seat stabilizing back and head, Multiseat Stablio System car seat. Please read the instructions thoroughly before using the product.

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The seat stabilizing back and head, Multiseat Stablio System car seat

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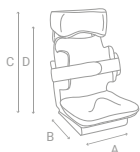
The seat stabilizing back and head, Multiseat Stablio System car seat is designed for children and adults with cerebral palsy. It compensates and alleviates disabilities by stabilizing the patient in a sitting position. The product consists of three Stablio Multiseat system cushions attached to a metal frame.

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CAR SAFETY SEAT DESIGNED FOR CHILDREN IN THE AGE GROUP III, 22-36 KG

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The holding device for children belongs to the category “universal”. It obtained UNECE type approval certificate (according to regulations n. 44, series of amendments 04) for general use in vehicles and is suitable for installation on the majority of car seats. The proper installation is possible if the manufacturer of a vehicle declared in a car instruction manual that the vehicle is suitable for installation of the holding device for children (category “universal” for this age group). This holding device for children was classified into the category „universal” in more severe conditions when compared to conditions applied to the previous projects, which are not affixed with this information. In case of doubt one should consult with the manufacturer or retail seller of the device.

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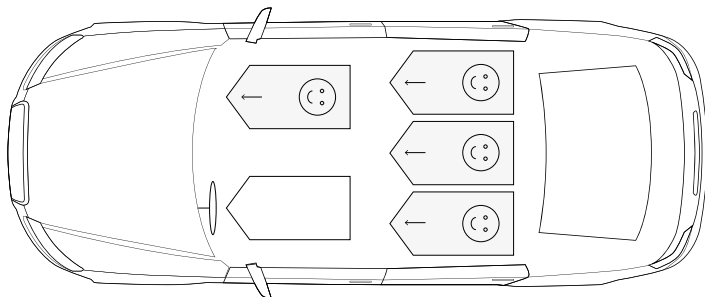
WARNING

In order to properly secure a child one should definitely keep to the indications referring to mounting and using of the device, which are contained in this manual.

USE IN A VEHICLE

Car safety seat Stabilo Multiseat is approved to be mounted with 3-points seat belt, either on a front or on a back car seat, **but exclusively forward-facing**.

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
DANGER

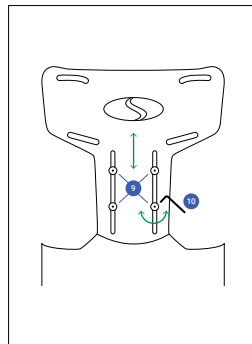Under no circumstances it is allowed to fasten a child with 2-points seat belt. In case of accident a child may suffer from serious, even fatal injuries. The safety car seat may be used solely in vehicles with originally installed, 3-points seat belt equipped with a retractor in conformity with the norm ONZ/EKG no 16, or other equivalent norm.

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CAR SAFETY SEAT STRUCTURE

1. Seat
2. Backrest
3. Headrest
4. Seat belt lug
5. Seat belt leading signs
6. Seat belts fastening scheme
7. Metal frame
8. Adjustable part of the headrest
9. Regulating bolts
10. Wrench for screwing the bolts.

The safety car seat is composed of the three Stabilo System cushions (1, 2, 3) mounted on the load-bearing metal frame. Each of the cushions enables to precisely adjust to a child body shape. A special vacuum pump serves to obtain a proper shape. The pump is an integral part of the safety car seat.

HEADREST HEIGHT ADJUSTMENT

The headrest should be adjusted in such a way to leave ap. 4 cm wide room between a child's arms and the headrest (approximately two fingers wide). The height of the headrest's position is regulated with regulating bolts (9) in the back part of the safety car seat. We unscrew 4 bolts with a special wrench which goes with the safety car seat - one anticlockwise turn. We adjust a demanded headrest's height. Then we screw the regulating bolts in clockwise. Use only the attached wrench for screwing the bolts (10).

PRODUCT SAFETY WARNINGS

- do not leave a child unattended
- the luggage and other heavy or sharp objects should be properly secured in a car
- the safety car seat should be always mounted with security seat belts, even if we don't carry a child in a car
- it is obligatory to regularly check the safety car seat paying special attention to its technical condition

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
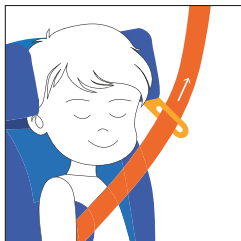
3

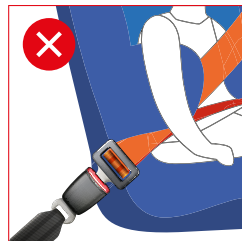4

- if you not sure do not use
- protect the safety car seat from intensive sunrays
- remember that on hot days the safety car seat may be strongly heated up. In order to protect your child from burns the safety car seat can be covered, and before seating a child a car may be cooled down and the temperature of the car safety seat's padding checked hard elements and stiff safety car seat's parts should be placed and installed in such a way that they are not exposed to blockage by sliding door or seats of a vehicle in typical exploitation conditions
- **WARNING** it is forbidden to introduce any alterations in the construction of the safety car seat and to make any own repairs
- **ATTENTION** The safety car seat can not be used without the complete padding (Stabilo system cushions). The cushions are specially designed for the safety car seat and can not be replaced by other equipment
- **WARNING** It is not allowed to use the safety car seat when an accident occurred, even if there are no visible signs of damage

CHILD FASTENING IN THE SAFETY CAR SEAT

The safety car seat is mounted and secured together with a child by the 3-points security seat belt. Place the safety car seat on the car seat, trying to maximally press it to the car seat's backrest. Seat a child in the safety car seat. Fasten the 3-points security seat belt according to undermentioned rules:

A. Pull the security seat belt out and lead it in front of a child to the lock of the security seat belt. Stick the buckle to the lock of the security seat belt. You should hear a characteristic click. Place the crosswise part of the security seat belt over a child's hips in such a way that the belt goes through the places indicated by the red stripes on the safety car seat (1).

DANGER Make sure that the crosswise part (hip part) of the security seat belt runs as low as possible and is tight.

B. Put the upper part of the security seat belt diagonally over a trunk of a child. Insert the diagonal security belt into the special lug of the headrest of the safety car seat. Pull the diagonal belt diagonally up (2).

C. **ATTENTION** The lock of the security seat belt can not go between the seat part and the backrest part of the safety car seat (3, 4).

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DANGER Check if a seat belt is tight, and fits to a child's body.

A loose security seat belt does not provide a proper protection. Check if a seat belt is not twisted or jammed. Additionally, check if the buckle of the belt is stick. **If you need unfasten a child quickly, please press the button which opens the lock of the security seat belt and then pull it back, releasing a child.**

PRODUCT SAFETY WARNINGS

- Multiseat Stabulo System Cushion must be securely fitted to the metal frame
- do not install the cushion together with your child. Multiseat Stabulo is unstable seat without installation
- after the seat has been installed, check stability of seat or chair with the installed cushion
- never leave a child fastened to the seat unattended
- do not use the metal frame as a separate seat
- due to the therapeutic qualities, the seat must be shaped and used under the supervision of a doctor or a physiotherapist
- before each use, check all the screws in a metal frame are tight
- use of airy cover is advisable if increased ventilation is required around the body
- avoid contact with sharp objects that may damage the cushion
- do not clean using solvent-based cleaners, e.g. toluene or acetone
- do not pull on valve
- do not overinflate cushion
- use only special pump included with cushion for inflation
- do not iron
- in case of tear, micro-granules may escape which may be dangerous for small children

ATTENTION

- the thickness of the cushion can naturally reduce by 5% in the initial period of use
- in case of considerable changes of temperature use pump again to obtain proper hardness (higher temperature softens the cushion, lower temperature hardens the cushion)

INSTRUCTIONS FOR USE

The construction and functionality of Stabulo cushions allows our clients to mold them to the patient's body and also to set the required seat upholstery. The cushions shouldn't be harden to much due to the comfort of the child. Every cushion can be molded and hardened separately in the following way:

Hand pump

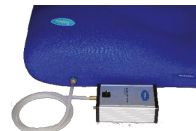
1. take out tube from box with electric pump
2. screw off valve cap, screw tube (tapped part) into cushion valve, and push second tube into SOFT socket in electric pump. Turn on pump and wait until cushion will be soft enough, then turn pump off
3. start shaping cushion (either manually to adjust shape or by sitting or leaning on it)

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- change position of tube from SOFT to HARD and turn pump on removing air until the desired hardness has been reached
- screw off tube, put cap back on and tighten
- to change the shape repeat steps from 2 to 5

Electric pump

- take out tube from box with electric pump
- screw off valve cap, screw tube (tapped part) into cushion valve, and push second tube into SOFT socket in electric pump. Turn on pump and wait until cushion will be soft enough, then turn pump off
- start shaping cushion (either manually to adjust shape or by sitting or leaning on it)
- change position of tube from SOFT to HARD and turn pump on removing air until the desired hardness has been reached
- screw off tube, put cap back on and tighten
- to change the shape repeat steps from 2 to 5

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INSTRUCTIONS FOR USE

The headrest should be adjusted in such a way to leave approx. 4 cm wide room between a child's arms and the headrest (approximately two fingers wide). The height of the headrest's position is regulated with regulating bolts in the back part of the safety car seat. We unscrew 4 bolts with a special wrench which goes with the safety car seat – one anticlockwise turn. We adjust a demanded headrest's height. Then we screw the regulating bolts in clockwise. Use only the attached wrench for screwing the bolts.

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Find Stabilo System Cushions on:

- washing in a washing machine is possible in up to 40°C. There is no need of using a washing powder, a fabric softener is sufficient
- wipe dry with a paper towel
- for disinfection you can use typical disinfectants used in hospitals
- do not use solvent-based cleaners such as toluene or acetone

WARRANTY

- the manufacturer gives a 24-months guarantee
- ensures the highest quality, proper operation and safety in accordance with the ECE 44/04 standard when used in accordance with the intended use and the rules contained in this manual
- each element of the seat has an individual and unique serial number
- the guarantee does not exclude, suspend or limit the rights of the buyer resulting from the non-compliance of the goods with the contract
- the guarantee does not cover mechanical damage of the cushion
- in case of any defects contact the seller with the proof of purchase
- if in doubt, please contact the seller or the manufacturer directly
- all incidents should be reported to a seller or to the manufacturer directly

DECLARATION OF CONFORMITY

- the client made his own alteration or repairs
- the damages are a result of an accident
- the warranty concerns the wear and tear including the changes of the color or fading of the elements
- applicant does not have a proof of purchase

DECLARATION OF CONFORMITY

The seat stabilizing back and head, Multiseat Stabilo System car seat is in accordance with the European Union directive EEC/93/42 and Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices.

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